



**S O U T H E R N C R A F T**™

D E N T A L L A B O R A T O R Y , C D L

Precision Crafted. Technology Driven.

## AESTHETIC CHECK LIST

To help us serve you better, we have put together a check list of the items needed to complete an aesthetic case.

1. Photos of pre-op teeth with shade guide, 1:1, 1:2 from different angles.
2. Photos of preps with stump shade tabs.
3. Photos of provisionals (not numb) including: eyebrow-to-chin natural smile, 1:2 retracted, profile smile.
4. Photo of stick bite (eyebrow to chin).
5. Stick bite (horizontal plane parallel to the horizon).
6. Model of pre-op.
7. Impression of provisionals. Gum tissue capture here is as important as margin capture in an impression of the preps.
8. Bite registration.
9. Detailed lab prescription thoroughly filled out.

HERE ARE A COUPLE OF HINTS WHEN SENDING A CASE:

1. Don't send models. Send polyvinyls.
2. Label all sides and photos with the doctor's name and patient's name.
3. Make sure the models are securely packed or they will break.
4. Securely pack transfer jigs. If they move, they become useless.

This check list is necessary to help us serve you and your patients better.