



S O U T H E R N C R A F T™

D E N T A L L A B O R A T O R Y , C D L

Precision Crafted. Technology Driven.

FULL MOUTH BITE RECOVERY

This is a procedure that will accurately duplicate the patient's exact bite. This will enable us to deliver you a case that will make your seat appointment hassle-free.

The Full Mouth Bite Recovery will need to be done in the sequence outlined below. The bites should be marked "left" and "right" and packed separately (upper prep to lower provisional, upper prep to lower prep, upper provisional to lower prep).

1. Upper Prep to Lower Provisional: Remove the provisionals on the upper second molars and upper centrals to get a tripod (three bites) of upper prep to lower provisional.
2. Upper Prep to Lower Prep: Remove the provisionals on the lower second molars and lower incisors to get a tripod, of prep to prep.
3. Upper Provisional to Lower Prep: Replace upper provisionals on both centrals and both second molars to make the final tripod, marked "left" and "right," of upper provisionals to lower preps.

